HUMANTECHNIK

scalla¹

D

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Seite 2

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We would like to congratulate you on the purchase of your **scalla¹** telephone and we hope it will give you many years of reassuring operation. Please read through this instruction manual carefully before operating your new telephone. They tell you how to set up the system correctly and explain all the systems features to you.

Special features

- Ring tone adjustable in volume and melody
- Bright visual incoming call signal by a white flashing light
- Speaker phone adjustable in volume and tone
- Extra-large illuminated numerical buttons
- 10 speed-dial buttons
- 8 photo speed-dial buttons with color indicator and inlay card on telephone receiver
- Hands-free function
- Mute function
- Hearing-aid-compatible
- Connection jack for headset with 2.5 mm connector
- Connection jack for teleloop, earphones, etc. with 3.5 mm connector
- Connection jack for vibrating pillow and/ or modules (see page 25)

Safety instructions

To enable you to get your telephone working quickly and to use it safely, it is essential that you read the following safety instructions and the section on „Getting started“.

- **Attention:**

Your telephone can be set to a high volume. To protect your hearing or the hearing of people with normal hearing, particularly of children, from any damage caused by excessively loud volume, please observe the following information:

1. Set the volume to low before you place the telephone receiver on your ear.
 2. While using the device, set the volume to a level that is comfortable for you.
- Place the telephone on a non-slip surface.
 - Lay the connecting cable safely in order to avoid the risk of accidents.
 - Never expose the telephone to any of the following influencing factors:
 - sources of heat
 - direct sunlight
 - moisture
 - Protect your telephone from damp, dust, aggressive liquids and steam.
 - Make sure you plug the wires to the corresponding jacks only.
 - Only connect approved accessories to it.
 - Never
 - open the telephone casing yourself!
 - carry the telephone by the connecting cables!

Contents	page		
Getting started		Settings	
Standard components	19	Setting the call signal volume	25
Telephone connection	19	Additional jacks	
Buttons and what they mean	19	Connection jack for vibrating pillow and/ or modules	25
Controls		Headset jack	25
Controls on the front/ the left	20	Headphone jack	25
Controls on the back/ the right	21	Wall mounting	26
Using the telephone		If your telephone doesn't work...	
Making calls	22	General information	27
Answering calls	22	The telephone doesn't work	27
Using the telephone with a headset	22	Maintenance and care	27
Using the speaker phone during a telephone call	22	Warranty	27
Setting the volume for the speaker phone	22	Environmental note/ Waste disposal	28
Last number redial	22	Technical data	28
Receiver volume / treble / bass control settings	22	CE-symbol	28
Switch to reset the telephone volume	23		
Using the telephone with a hearing aid	23		
Mute ()	23		
Inserting a dialing pause (PABX)	23		
Inquiry button ()	23		
Speed-dial buttons			
Assigning numbers to the speed-dial buttons	24		
Dialing with speed-dial buttons	24		
Photo speed dial buttons	24		

Getting started

Standard components

Please check if all following components are included:

- Telephone base unit
- Telephone receiver
- Receiver cord
- Telephone cable
- 7.5V plug-in PSU
- Speed-dial inlay card
- Operating instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Telephone connection

1. Insert the receiver cord (spiral-shaped cable) into the jack marked with a receiver symbol on the left side of the base unit and into the receiver.
2. Insert the smaller plug (western plug) on the telephone wire into the  jack (rear panel). Insert the larger plug (TAE plug) into the outlet for your telephone connection.
3. Insert the plug-in mains adapter cable into the "DC 7.5V" jack. Plug the mains adapter into a 230 V mains socket.

Warning!

Don't place the telephone close to televisions, VDUs, fans, etc. so as to minimize the risk of interference. Don't place the telephone in direct sunlight or close to a source of heat.

Only use the original telephone cable or a comparable cable with an international configuration.

Buttons and what they mean

 Speed-dial buttons
 (Telephone number memory)

 Programming button
- Storing telephone numbers

 Mute button
- Switching the microphone to mute
- Select ring tone while receiving an incoming call

 Speaker button:
- "Hands-free function": engage the line, hang up
- Enable/ disable "conference" mode during a call

 Headset button
- Using telephone with a headset
- Line busy or disconnect

 Amplification on/ off

 Inquiry button, e.g. for putting calls through if the telephone is connected to a telephone system.


 Redial last number/ Insert pause while dialing a number

 Photo speed dial buttons
Buttons with space for photos


 Volume keys:
- Selecting the volume of the ring tone
 - Selecting the volume of the phone speaker

Controls


Controls on the front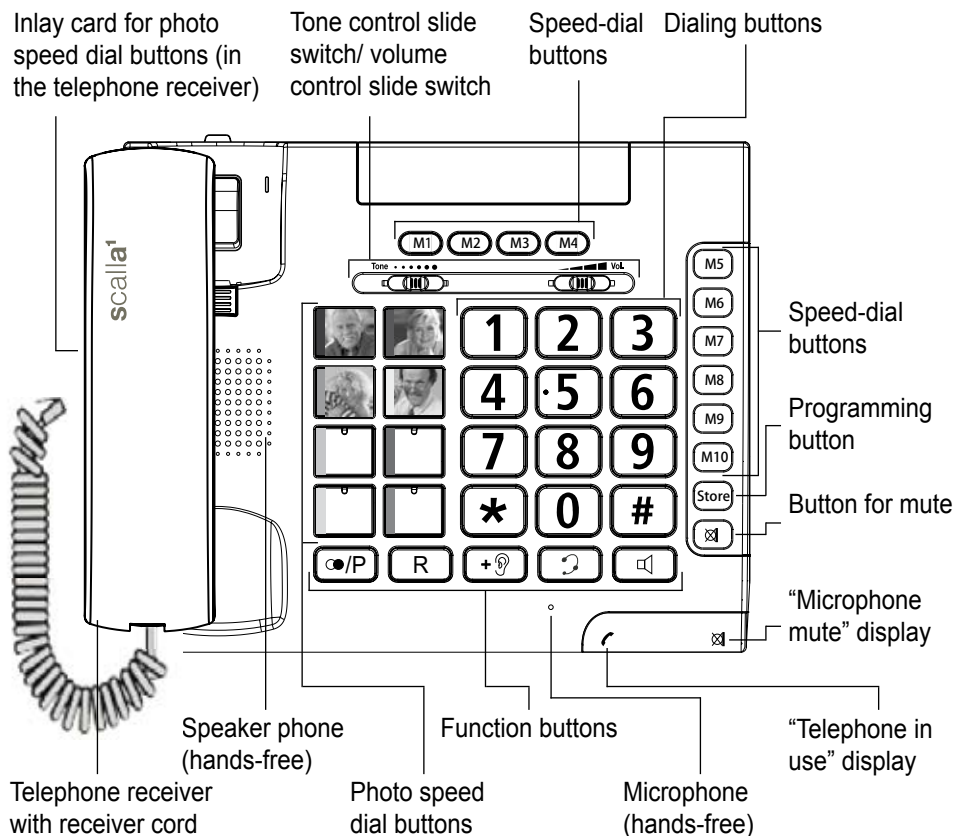


Controls on the left

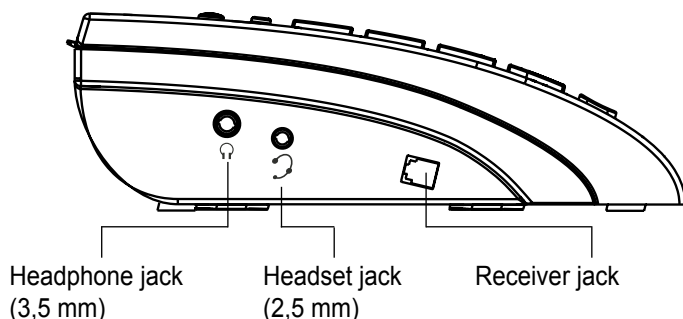Controls on the back

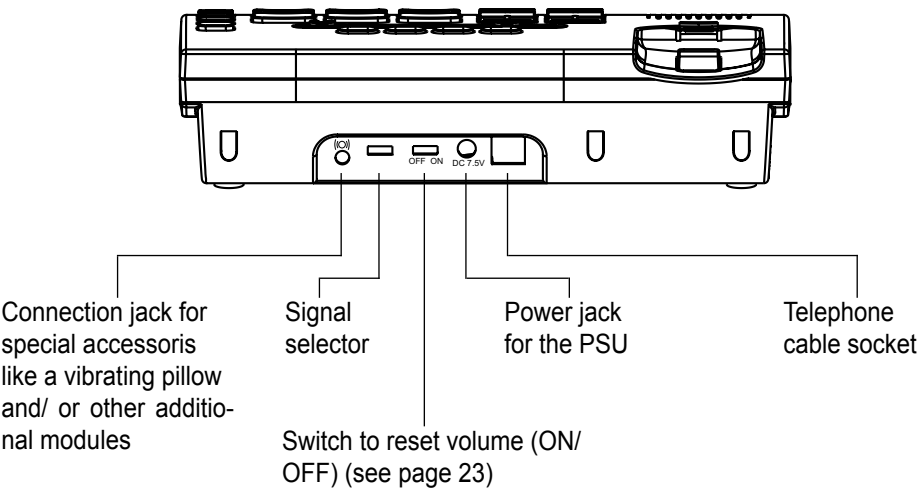Controls on the right

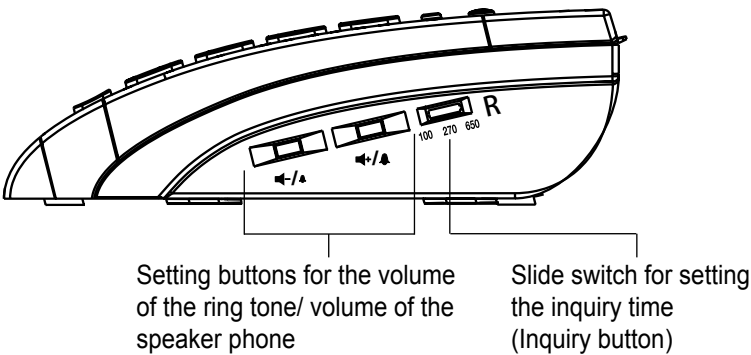Using the telephone

Making calls

1. Lift up the receiver or press the  button.
2. Dial the telephone number.
3. In order to end the call, replace the receiver or press the  button again.

Answering calls

1. The telephone rings.
2. Lift up the receiver or press the  button.
3. In order to end the call, replace the receiver or press the  button again.

Using the telephone with a headset

- Making calls

1. Press the  button.
2. Dial the telephone number.
3. In order to end the call, press the  button again.

- Answering calls

1. The telephone rings.
2. Press  to conduct the call over the headset.
3. In order to end the call, press the  button again.

Using the speaker phone during a call

1. Press the  button to activate the telephone speaker during a call. The receiver speaker remains activated.
2. Now replace the telephone receiver on its hook so that the speaker phone switches on.

3. You can continue with your call using the speaker phone and the integrated microphone.
4. Press the  button to deactivate the telephone speaker again.

Note:

A call is ended if you press the  button while the receiver is on the cradle.

Setting the volume for the speaker phone

In hands-free operation, you can use /▲ or ▲/▲ on the right-hand side of the telephone to adjust the volume.

Last number redial

1. Lift up the receiver.
2. If you press the  button, the last number called is redialed.

Receiver volume/ treble/ bass control settings

The receiver volume can be adjusted during a call with the "Volume" slide switch on the front side of the telephone (see Fig. 1, page 23).

Additional amplification can be switched on by pressing the  button. When additional amplification is activated, the receiver volume can be raised up to 40 dB. A status indicator will light up on the telephone when the amplifier is switched on. The additional amplification is deactivated by pressing the  button again.


Use the tone control (sliding switch on the front, see Fig. 1) to adjust the tone.

Note:

The tone control will not function unless the amplifier is switched on.

Fig. 1:

Tone control/ volume control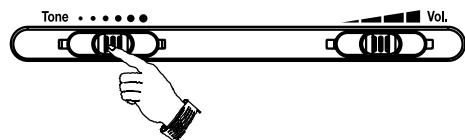

Switch to reset the telephone volume

There is a switch on your telephone that automatically resets the volume to the original level when you hang up the receiver. This means that people who don't need the amplification can use the telephone without having to worry that the volume may be too loud and hurt their ears. To select this as the normal mode, set the switch +Ⓜ on the back of the telephone to "OFF".

If you are the only one who uses the telephone or you would like to deactivate the reset function, move the switch +Ⓜ on the back of the telephone to the "ON" position. Now the volume will not be reset when you hang up and all your settings will be active for your next telephone call.

Using the telephone with a hearing aid

You can use your telephone with hearing aids that are equipped with a telephone coil (T-coil). To do so, set the selection switch on your hearing aid to "T" or "MT". When using the telephone, please be sure to hold the receiver as close to your hearing aid as possible.

Mute (☒)

During a telephone call, you can also temporarily switch off the telephone microphone so that the person you are talking to can no longer hear you.

Simply press the key ☒ marked to switch the microphone off.

Inserting a dialing pause (PABX)

Press ☐/P to interrupt the dialing process for a preset time.

Inquiry button (R)

This function is required in order to activate certain T-Net features. If you press the R button, the telephone line is interrupted for a certain length of time (100, 270 or 650ms).

Speed-dial buttons

Assigning numbers to the speed-dial buttons

Any telephone number (a maximum of 32 digits) can be assigned to the 10 speed-dial buttons and the 8 photo speed-dial buttons.

1. Lift up the receiver or press the  button.
2. Press the  button.
3. Enter the "telephone number" you require.
4. Press the  button again.
5. Press one of the ..., ... buttons or a photo speed-dial button to assign the telephone number you have entered to that button.
6. Replace the receiver or press the  button again

Dialing with speed-dial buttons

Lift up the receiver or press the . Press one of the ..., ... buttons or a photo speed-dial button to dial the telephone number you have entered for that button.


Note: If the telephone is disconnected from the mains/ telephone network for more than 60 minutes, the stored telephone number will be deleted.

Photo speed-dial buttons


For these buttons it is possible to insert a small photo of the party you wish to call. In addition, you can assign a name to each picture with color coding and the inlay card in the telephone receiver.

Example:

Two photo buttons with pictures of the party to be called

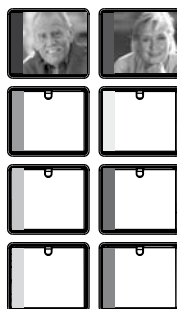Naming the party to be called on the insert plate in the telephone receiver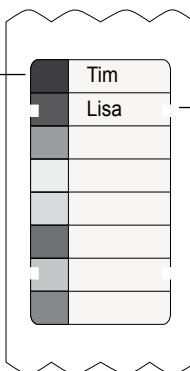

Settings

Setting the call signal volume

The telephone's ringer volume can be set by means of the both buttons  and  on the right side of the telephone.

Additional jacks

Connection jack for vibrating pillow and/or modules

Your telephone is equipped with a connection jack for a vibrating pillow or other additional signaling modules. You can take advantage of this option to signal an incoming call not only with a very loud ring and the very bright flashing light, but also with an optionally available vibrating pillow and/ or other module.

In addition to the acoustic ring tone, you can choose whether you also want incoming calls to be signaled visually, with vibration or with both. To select the desired type of signaling, slide the signal switch (on the back of the telephone) to one of three possible positions:

- Switch left: Visual with signals via a connected vibrating pillow and/ or module
- Switch in the middle: Signal via a connected vibrating pillow and/ or module
- Switch right: Visual only

The following optional modules can be connected to the telephone:

- Flash module
- Switch module
- Acoustic module

Headset jack

Standard headset port with a 2.5 mm jack (for connecting a PL-100/ PL-200, for example).


Headphone jack

Headphone output (3.5 mm jack plug) to connect a headphone, teleloop or similar.

Wall mounting

1. Push the holder for the telephone receiver out of its position and turn it by 180 degrees. Push the holder back into its original position so that the hook is pointing upwards (see Fig. 2). This prevents the receiver from falling from the telephone base when the telephone is mounted on the wall.


Fig. 2:
Holder for the telephone receiver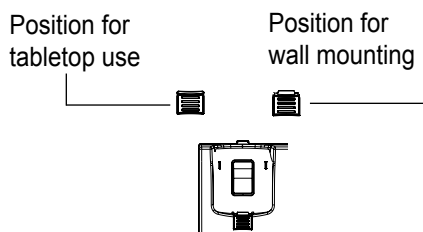

2. Screw in two screws (not included) one directly above the other 8 cm apart at the place you would like to mount the telephone on the wall.

3. Now set the telephone base on the screws and slide it down slightly to secure it in place (see Fig. 3).

Fig. 3:
Wall mounting for the telephone

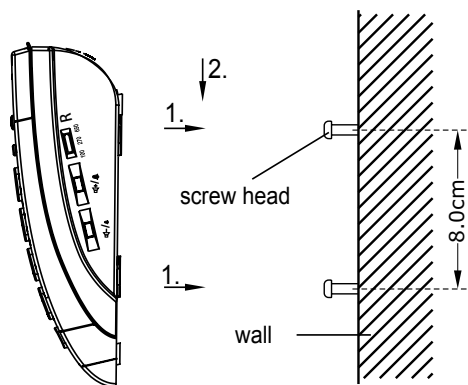If your telephone doesn't work...

General information

Make sure that the telephone cable isn't damaged and is correctly inserted.

Disconnect any additional equipment, extension cables and other telephones.

If the telephone works then, this indicates that the fault is caused by another piece of equipment.

Try to connect your telephone to a neighbour's line, for example. If it works there, this indicates that there may be a fault on your telephone line. Call the service provider to report the fault.

The telephone doesn't work

- Make sure that the mains adapter is correctly inserted into the telephone and into the mains socket.
- Make sure that the telephone cable is correctly inserted into the telephone and into the telephone socket.
- Connect another telephone to your telephone line. If it works, it indicates that your telephone is probably faulty.
- Only use the original cable or a telephone cable with an international configuration.

Maintenance and care

The telephone is maintenance-free. If the telephone does become dirty, simply wipe it clean with a soft, damp cloth. **Note: remove the plug-in PSU from mains socket!**

Never use alcohol, thinner or other organic solvents. The telephone should never be exposed to sunlight for a long time and should also be protected from intense heat, moisture or strong mechanical vibrations.

Important Notice

This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The telephone is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

Environmental note/ Waste disposal

Disposal of used electrical and electronic equipment (applicable in the countries of the European Union and in other European countries which operate a separate collection system for these devices).

The symbol on the product or its packaging indicates that this product should not be treated as normal household waste. It should be handed in to a collection center for recycling electrical and electronic equipment. By playing your part in ensuring the correct disposal of this product, you will be safeguarding the environment and the health of other people. The environment and health are jeopardized by incorrect disposal. Material recycling helps to reduce the consumption of raw materials. You can obtain further information about recycling this product from your local council, from local waste disposal operators and from the shop where you bought the product.

Technical data

Power supply:

Plug-in PSU:

Primary 220/230V 50 Hz AC

Secondary 7,5V DC/ 500 mA

Dimensions (LxWxH):

243 mm x 195 mm x 94 mm

Weight:

ca. 870 g

CE symbol

This telephone satisfies the following EU directives:

- 2002/95/EG RoHS directive
- 2002/96/EG WEEE directive
- 1999/5/EG R&TTE directive
- 2004/108/EG EMC directive
- 2006/95/EG Low voltage directive

Compliance with the directives listed above is confirmed by the CE seal on the telephone. CE compliance declarations are available on the Internet at www.humantechnik.com.

Technical specifications subject to change without notice.

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Humantechnik GmbH
Humantechnik GHL AG